


AMSCO 2080 TABLE (ORTABLE16)

TO RAISE or LOWER TABLE --

1.) Push down on FOOT PEDAL labeled "RAISE" or "LOWER", and keep depressed until table is at the desired height.

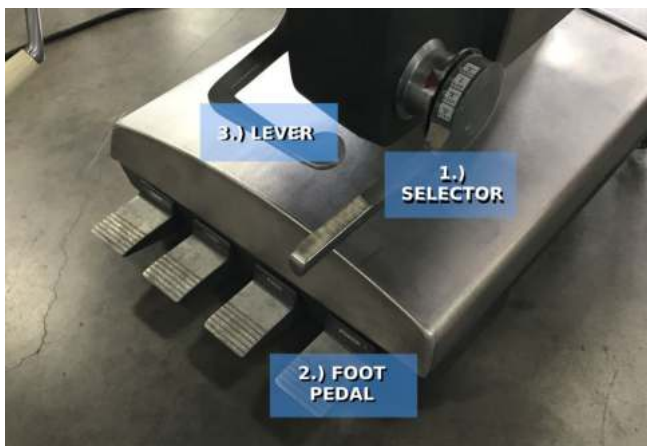

TO RAISE/LOWER BACK BED --

1.) Move SELECTOR (1) until ARROW points to "BACK"

2.) Push down on the FOOT PEDAL (2) labeled "POWER" and keep depressed while you...

3.) Push down/pull up on LEVER (3) underneath the table until bed is at the desired angle.

TO RAISE/LOWER LEG BED --

1.) Move SELECTOR Until arrow points to "FOOT"

2.) Push down on the FOOT PEDAL (2) labeled "POWER" and keep depressed while you...

3.) Push down/pull up on LEVER (3) underneath the table until bed is at the desired angle.

TILT BED SIDE TO SIDE --

1.) Move SELECTOR Until arrow points to "SIDE"

2.) Push down on the FOOT PEDAL (2) labeled "POWER" and keep depressed while you...

3.) Push down/pull up on LEVER (3) underneath the table until bed is at the desired angle.

[Click HERE](#) to view item details on The Alpha Companies website.