

GAMBRO PHOENIX DIALYSIS

Instructions for Demo Mode

A video walk-through of these instructions can be viewed at

<https://youtu.be/ufMCj7z7BkM?t=220>

How to Power on and start "Demo Mode"

Step 1.) Turn machine on (The switch is located on the right hand side of machine.)

Let it boot up.

Step 2.) Hit the SET UP button on the right side of the screen.

Wait as it sets up until the PRIME button appears.

Step 3.) Hit the PRIME button, then hit the PRIME W/O UF button.

Machine will start circulating (with either saline or blood. It will run like this for 2 minutes then it will stop.

Step 4.) Repeat step 3- hit the PRIME button and then hit the PRIME W/O UF button.

When it is done priming, wait for the RECIRCULATE button to appear.

Step 5.) Hit the RECIRCULATE button. This will make the machine run for 55 minutes.

Repeat as needed.

To Power Off

Step 1.) Simply turn off the machine with the power switch on the right hand of the machine.

[Click to HERE to view item details on Alpha Props website](#)

