

## **HOSPIRA SYMBIQ DUAL CASSETTE**

### Instructions for Basic Operation

A video walk-through of these instructions can be viewed at <https://youtu.be/YJQN3wG-g88>

#### **How to Power on and put into "delayed mode"**

Step 1.) Press WHITE power button on.

Once it boots the cassette(s) will open and then close again automatically.

Step 2.) On New Patient? dialog, press YES and then CONTINUE.

Step 3.) On CCA, Press SELECT CCA and then CATH LAB.

Step 4.) For Caregiver ID, type 111 and press ENTER.

Step 5.) For Last Name, type AAA and press ENTER.

Step 6.) For First Name, type BBB and press ENTER.

Step 7.) For ID, type 222 and press ENTER.

Step 8.) Press ENTER.

Step 9.) Touch CLOSE.

Step 10.) Press A.

Step 11.) Press SELECT INFUSION and then IV FLUID.

Step 12.) Press 1,000 ML and if you are asked about DRUG DUPLICATION press CONTINUE.

Step 13.) Select RATE and type 1 and press ENTER.

Step 14.) Select VTBI and type 100 and press ENTER.

Step 15.) Press NEXT A.

Step 16.) Press DELAYED START

Step 17.) Select HOURS and type 11 and press ENTER

#### **Repeat steps 10 through 17 for Side B**

Step 18.) Press SETTINGS

Step 19.) Press BRIGHTNESS

Step 20.) At AUTOMATICALLY DIM BACKLIGHT WHEN ON AC? select NO.

Step 21.) Press SAVE and then EXIT.

#### **To Power Off**

Step 1.) Touch Screen

Step 2.) Press STOP BASIC

Step 3.) Press WHITE power button.

Step 4.) Press OK

[Click to HERE](#) to view item details on  
[Alpha Props website](#)

